



Anti-aging & photoaging treatment for the face

A R T Jalupro 3° class Medical Device CE N° 0373

Period of treatment (T=tempo)



Notes: It is suggested Jalupro use after 5° week for the maintenance (one intradermal implant a month).

A R T Proglyme (Tiny drinkable bottles): 1 tiny bottle a day.



Notes: 1 tiny drinkable bottle a day for 3 months.

A R T Intensive anti-wrinkle & post peeling serum and/or Anti-wrinkle face cream: 2 times a day



Notes: Serum and/or cream 2 times a day for 4 months. Serum combined with cream for dry skin, while it can be used alone on the normal and oily skin (see leaflet).

A R T Intensive anti-wrinkle facial mask: 2/3 times a week



Notes: Facial mask 2/3 times a week for 4 months. Apply it without massaging and leave it on 2 hours. Cleanse face with lukewarm water to remove (see leaflet).

A R T Key



Treatment pause period



Period of treatment



Other treatment

Laser Resurfacing, IPL, Peeling (chemical and mechanical peeling) treatment

A R T Jalupro 3° class Medical Device CE N° 0373

Period of treatment (T=tempo)



A R T Proglyme (Tiny drinkable bottles)



Notes: 2 tiny drinkable bottles a day before the operation for 1 week. 1 tiny bottle a day after the operation for 8 weeks.

A R T Intensive anti-wrinkle & post peeling serum and/or Anti-wrinkle face cream: 2 times a day



Notes: Serum or cream 2 times a day for 9 weeks. Serum combined with cream for dry skin, while it can be used alone on the normal and oily skin (see leaflet).

A R T Intensive anti-wrinkle facial mask: 2/3 times a week



Note: Facial mask 2/3 times a week for 8 weeks. It is better not to use the mask during the first week after peeling.

A R T Key



Treatment pause



Period of treatment



Other treatment




Thermage treatment

A R T Jalupro 3° class Medical Device CE N° 0373

Period of treatment (T=time)




A R T Proglyme (Tiny drinkable bottles)

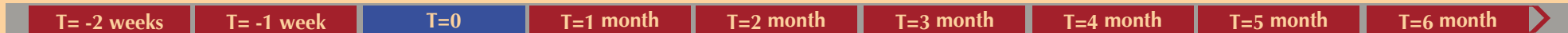


Notes: 2 tiny bottles a day for 6 months

A R T Intensive anti-wrinkle & post peeling serum and/or Anti-wrinkle face cream: 2 times a day

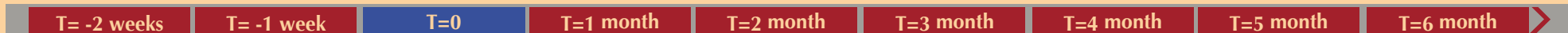

Notes: Serum and/or cream 2 times a day. Serum combined with cream for dry skin, while it can be used alone on the normal and oily skin (see leaflet).

A R T Intensive anti-wrinkle facial mask: 2/3 times a week

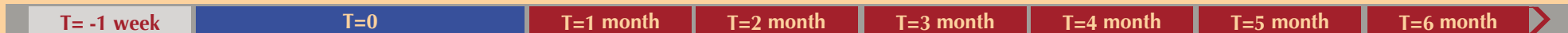

Notes: Facial mask 2/3 times a week

A R T Legenda



Treatment pause



Period of treatment



Thermage