

AMINOGAM

Adjuvant Gel in the regeneration process of the oral mucous

DO NOT USE THE PRODUCT FOR APPLICATIONS OTHER THAN THOSE SPECIFICALLY
INDICATED IN THIS LEAFLET

COMPOSITION

Ingredients

Purified water, Sodium hyaluronate, Glycine, L-Proline, L-Leucine, L-Lysine Hcl,
Methyleparahydroxibenzoate, Propyleparaben, Sorbitole, Polyvinyl pyrrolidone, Sodium hydroxide.

DESCRIPTION

VULNAMIN is a gel of non-animal origin that is highly biocompatible and that can be used as an adjuvant to facilitate the healing of oral-gingival tissue with a protective action in cases of:

- a) Oral Surgery (protection of surgical and suture wounds)
- b) Extractive surgery (regeneration of the alveolous and prevention of post-extractive alveolitis)
- c) Regenerative, parodontal and implant surgery.
- d) Pre-prosthetic and orthodontal surgery.
- e) Major and minor laser surgery
- f) Orthodontic and prosthetic traumatic situations
- g) Protection of vital tooth stumps in patients with fixed prostheses or with dental hypersensitivity
- h) Ulcerous boils of the oral mucous (auto-immune, vasculitic and infective).

AMINOGAM aids rapid regeneration of tissue and maintains the ideal moist conditions to help the re-epithelisation of the affected area.

PACKAGING

1 tube of 15 ml.

PRODUCT USE

AMINOGAM is an invasive medical device made in conformity with Regulation 93/42/CEE MDD; it is intended to help the cellular reintegration process, promoting fast healing.

The reconstruction of the tissues and the oral-gingival mucous are aided thanks to its effects. It can also be used in post-operation treatment.

INDICATIONS FOR USE

Careful wash hands before each application, apply a layer of the gel on the exposed area, massage it with a finger so as to spread the product over the area. Repeat the treatment after each main meal ensuring that proper oral hygiene has first been carried out. Continue the application for 3-4 weeks until the symptoms have disappeared.

Do not take food or drink or wash out the mouth for at least an hour after applying the product.

COUNTER – INDICATIONS

- Hypersensitivity to any of the ingredients.
- No overdose phenomena are known.
- No side-effects are known.
- No side-effects with other drugs or medicinal substances are known.

AMINOGAM has high tolerability; nevertheless local reactions may sometimes occur, due to hypersensitivity which may disappear spontaneously.

It is advisable to consult a physician if the product is to be used for more than 30 days.

INCOMPATIBILITY

This product can be used together with anti-inflammatory drug use.

Although no secondary effects are known, in case of pregnancy and breast feeding, it is advisable to use the application only in accordance with a physician's advice.

WARNINGS

- Do not use if the packaging is open or damaged.
- Do not use after the use-by date has expired.
- Close the tube properly in order to maintain proper storage of the product.
- Do not store in moist conditions and keep away from sources of heat.
- Keep away from children.
- Do not sterilize.
- For external use only.

MANUFACTURER

Professional Dietetics S.r.l. – Via Ciro Menotti, 1/A – 20129 MILANO – Tel. 02744020 – Fax 0270008791

Sales Licensee (Italy) – ERREKAPPA EUROTERAPICI - Milano