

VULNAMIN

Adjuvant in the healing process: Cream

DO NOT USE THE PRODUCT FOR APPLICATIONS OTHER THAN THOSE SPECIFICALLY INDICATED IN THIS LEAFLET

COMPOSITION

Ingredients

Sodium hyaluronate, Glycine, L-Lysine Hcl, L-Leucine, L-Proline

Excipients

Purified water, Cetilstearyl octanoate, Acrylic acid Copolymers and vinylic esters, cetil stearilic alcohol, Potassium cetyl phosphate, Imidazolidinyl urea, Phenoxy ethanol-paraben, Sodium hydroxide

DESCRIPTION

VULNAMIN is a cream of non-animal origin that can be used as an adjuvant to facilitate wound healing in all conditions where surgery is not possible and where relevant dermal damage is accompanied by deep skin loss such as in chronic ulcerous wounds, severe burns and bedsores. It facilitates a faster regeneration of tissues and maintains the ideal moisture conditions to promote the re-epitelization of cutaneous lesions.

PACKAGING

1 tube of 50 ml.

PRODUCT USE

VULNAMIN is an invasive medical device made in conformity with Regulation 93/42/CEE MDD; it is intended to help the cellular reintegration process, promoting fast healing.

Depending on the depth and width of the wound, it promotes the reconstruction of connective tissue and therefore regeneration of epithelial cells.

INDICATIONS FOR USE

Dress the ulcer according to the wound width and on VULNAMIN absorption speed.

Alternative therapies include: astringent substances which may cause vasoconstriction, soaps which may cause edemas or dehydration, or rubs with alcohol which may cause damage by removing cutaneous tissue fats and make the skin drier and provoke fissures. VULNAMIN however promotes the elimination of the necrotic tissues, protects skin integrity and prevents the spreading of germs.

Keep the lesion under constant observation.

How often the dressing needs changing will depend on wound conditions. Initially, daily changes may be necessary. After a few days the dressing changes may be done every three days at most.

PRECAUTIONS FOR USE

Before dressing each wound, wash hands carefully.

Remove possible foreign bodies with plenty of saline by sprinkling from the centre towards the outside of the wound.

After cleaning the area, the wound can be plugged with a few sterilized gauzes to remove excess blood.

If the wound does not require further operations, apply VULNAMIN to promote healing process after cleaning the wound.

COUNTER - INDICATIONS

Hypersensitivity to any of the ingredients.

No overdose phenomena are known.

No interactions with drugs and medicinal substances are known.

VULNAMIN has shown high tolerability; nevertheless local reactions may sometimes occur, due to hypersensitivity which may disappear spontaneously.

It is advisable to consult a physician in case of concurrent pharmacological treatment.

INCOMPATIBILITY

Although secondary effects are not known, in case of pregnancy and breast feeding, it is advisable to use the application only in accordance with a physician's advice.

WARNINGS

- Do not use if the packaging is open or damaged
- Keep away from children
- Do not sterilize
- For external use only

MANUFACTURER

Professional Dietetics S.r.l. – Via Ciro Menotti, 1/A – 20129 MILANO – Tel. 02744020 – Fax 0270008791